

Alden Classic Sphere & Toric

Soft contact lenses, 38% water, (polymacon)




Features

- Recommended for those patients where a 'custom fit' is required.
- A wide range of available parameters enables the fitter to achieve a truly custom design for the hard to fit patient.
- The perfect replacement lens for your patients who prefer conventional lathe cut soft lenses, such as C.S.I.®, Hydron Mini™, and AquaFlex®
- Lathe cut to order from Polymacon (38% water)
- FDA approved for quarterly planned replacement and conventional use.

Parameters

Base Curves	7.70, 7.90, 8.10, 8.30, 8.50, 8.70, 8.90 mm
Diameter	13.00, 13.50, 14.00 mm
BV Powers	-30.00 D. to +30.00 D. in 0.25 D. steps
Cylinder (toric)	-0.50 D. to -6.00 D. in 0.25 D. steps
Axis (toric)	1° to 180° in 1° steps
Custom Parameters	Available, non-warranted only
Colors	Clear, Visi Tint Blue Aqua, Azure, Blue, Brown, Gray, Green, Jade, Yellow, Walnut colors are available in light (#1), medium (#2), and dark (#3)
Material	(polymacon) • 38% water • 9 Dk (Fatt Units) • daily wear

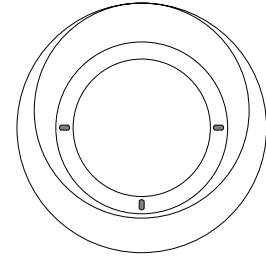

Toric Design

- + Prism Ballast
- + Thin Design (no prism in optics)
- + Round Bisymmetrical Optics
- + 3 marks at 0°, 180°, and 270°

Alden Classic Fitting Chart

Flat K-Reading	13.0 mm HVID=11.0	13.5 mm HVID=11.5	14.0 mm HVID=12.0
48.00 to 48.87	7.70	7.90	8.10
47.00 to 47.87	7.90	8.10	8.30
46.00 to 46.87	7.90	8.10	8.30
45.00 to 45.87	8.10	8.30	8.50
44.00 to 44.87	8.10	8.30	8.50
43.00 to 43.87	8.10	8.30	8.50
42.00 to 42.87	8.30	8.50	8.70
41.00 to 41.87	8.30	8.50	8.70
40.00 to 40.87	8.30	8.50	8.70
39.00 to 39.87	8.50	8.70	8.90
38.00 to 38.87	8.50	8.70	8.90

- Measure the Horizontal Visible Iris Diameter (HVID) and add 2 mm for the diameter selection. This lens will yield 1 mm of overlap. Custom diameters are available when HVID is less than 11mm or greater than 12mm.
- Measure the flat K reading, and locate it on the chart. Use the appropriate lens diameter column to find the suggested base curve.
- Convert the spectacle power in the major meridians to contact lens power using the vertex conversion table. Keep the cylinder axis the same as the spectacle axis.
- Use the "Tint Wheel" for color selection.
- Diameter and base curve suggestions are based on fitting experience and the most commonly used combinations. Certain corneal topographies may require a different diameter or base curve than is suggested.

Prices / Policies

Power	Single Warranted	Single Non-Warr.	3-Pack Non-Warr.
Spheres			
±10.00 D.	call for pricing		
±10.25 D. ±30.00 D.	call for pricing		
Torics			
±10.00 D.	call for pricing		
±10.25 D. ±30.00 D.	call for pricing		
Additional Tint Charges			
Standard Tint	call for pricing		
Standard Tint w/ Clear Pupil	call for pricing		
Prosthetic	call for pricing		

Lenses are available:

- Single, Warranted
- Single, Non-Warranted,
- 3-Pack, Non-Warranted

Warranted lenses are returnable for credit:

- 75% credit on exchanges
- 50% credit on cancellations
- Preferred Customers get 100% on exc./can.

Refer to Sales>Returns Policy for complete details.

C.S.I.® and AquaFlex® are registered trademarks of Ciba Vision Corp.
Hydron Mini™ is a trademark of Ocular Sciences, Inc.



Alden Optical Laboratories
13295 Broadway
Alden, New York 14004

web www.AldenOptical.com
email Info@AldenOptical.com
tele **800.253.3669**
fax **800.899.5612**

© July 2003
Alden Optical Labs.